Adriana Arce
Fon

arce@lgoptics.de
+49 (0) 22 26 / 15 85-18

Hermann Tondorf
Fon

tondorf@lgoptics.de
+49 (0) 22 26 / 15 85-17

Yvonne Brandt
Fon

brandt@lgoptics.de
+49 (0) 22 26 / 15 85-0

LightGuideOptics Germany GmbH
Heerstrassenbenden 10
D-53359 Rheinbach
– Germany –

Fon +49 (0) 22 26 / 15 85-0
Fax +49 (0) 22 26 / 15 85-20
E-Mail info@lgoptics.de
www.lgoptics.de

Minimal invasive laser-therapy
of varicose veins

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The appearance of abnormal leg veins is an endemic disease.

In Germany Approximately a population of 33 million show minor symptoms like veins modification or develop visible varices. More than one third has problems with cramps in the calf and pains.

Over 5 million people develop a venous insufficiency.

The creeping appearance of the symptoms is very dangerous. At the beginning the progress runs free of complaints and painless. Patients often do not annotate the diseases and they categorize first indices as innocent.

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Endovenous laser-treatment of the vena saphena magna and parva

The classical treatment for this disease – known as stripping – is quite painful; therefore new endovascular operation techniques emerged in the recent years. They provide minimal invasive approaches for the effective treatment of the disordered veins. The most advanced procedure in terms of efficiency and side effects is the laser treatment, which is approved in the USA and the European Union. It can be accomplished as outpatient treatment under local anaesthesia and offers a variety of advantages:

-  easy handling
-  low-pain treatment
-  short duration
-  procedure suitable for local anaesthesia
-  short convalescence
-  optimal cosmetic solution

The endovenous technology leaves no scars and minimizes the risk of infection. Soon after the treatment, the patient can resume his daily life.

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
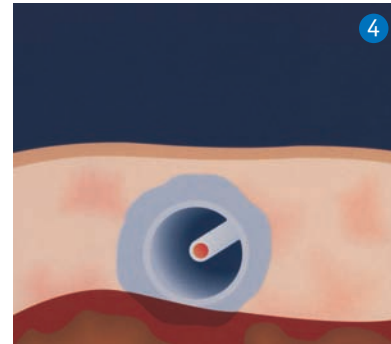
Characterisation of the vein structure with ultrasound

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
Protection of the surrounding tissue by tumescent anesthesia

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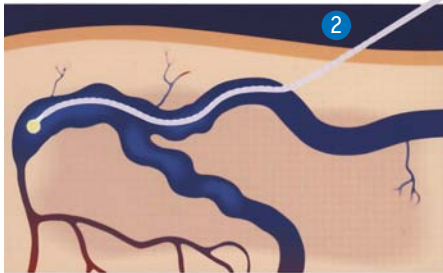
Positioning of the laser fiber for the endovenous treatment of varicose veins

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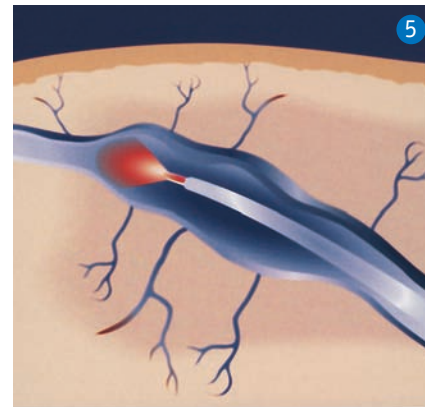Treatment with laser light

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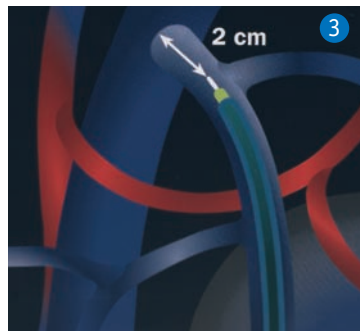Positioning of the fiber next to the saphenofemoral junction

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Endovenous treatment of the Reticular and Branch Varicosities

The endovenous laser therapy also allows the fast, controlled treatment of Reticular and Branch Varicosities.

This very efficient therapy has no known side effects, is well compatible and universally applicable:

Treatment:

- ✍ A very fine, special canula is injected into the vein after a surface anaesthesia. Even thin veins are easily treated with the very thin 200 µm fiber.
- ✍ Depending on the vein diameter laser light with a power range of 5 to 10 Watts is applied to the vessel
- ✍ After about seven days the treated Varicosities disappear. A subsequent treatment is typically not required.

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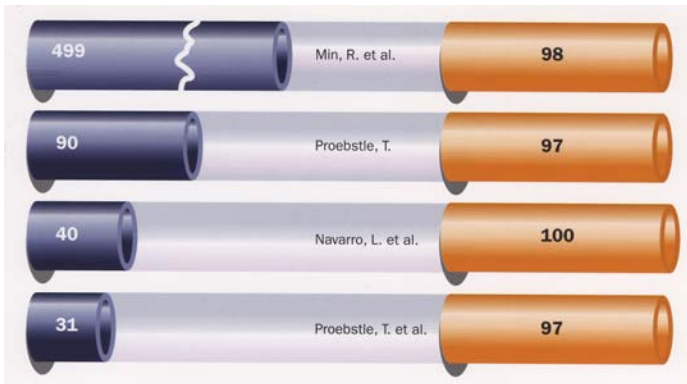
9 months after the treatment reduction of 81% for the diameter

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Proebstle, T., Gul ?, Kargl A, !{nop J.. Endovenous Laser Treatment of the Lesser Saphenous Vein with a 940-nm Diode Laser: Early results: Dermatol. Surgery 2003;29:357-361

Min RJ, Zimmet SE, Isaacs MN, Forrestal MD. Endovenous laser treatment of the incompetent greater saphenous vein. J Vasc Interv Radiol 2001; 12:1167-1171

Navarro L, Min RJ, Bone C., Endovenous laser: a new minimally invasive method of treatment for varicose veins--preliminary observations using an 810 nm diode Laser. Dermatol Surg. 2001; 2:117-22

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LGO-Vena Introducer Set

The area of application for the LGO catheter system is the minimal invasive treatment of the insufficient Greater Saphenal Vein and was particularly developed for this purpose. All components necessary for the operation are packed and kept together in a sterile bag.

The catheter is intended for to introduction into the Greater Saphenal Vein (in Seldinger technique). The canula provided in the set is used to puncture the vein and for the introduction of the guide wire, which is also contained in the set.

Markings in the distance of 1 cm make it easy to see the Insertion length. An adapter is attached to the laser fiber (LGO-Bare Fiber 600 μm), which is used to lock to fiber to the catheter and thus the fiber is distally 1 cm longer than the catheter.

An irradiation of the catheter is prevented.

The LGO-Vena Introducer set is a single use device.

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